

HAMA-SURF Spray™

Alcohol Free

Alcohol Free Surface Disinfectant Spray



For the surface disinfection of non-invasive medical devices, including surfaces sensitive to alcohol.



Mint Aroma

Key Product Features

Effective against a wide spectrum of organisms (see table opposite). Dentomint fragrance. Also contains detergent

Directions for use

Spray surfaces and objects to be disinfected from a distance of approx. 30 cm so that they are completely moistened and let the disinfectant take action over a period of 30 seconds. Allow surface spray to evaporate or wipe with a clean tissue or surface wipe.

Composition

Contains amongst other ingredients: Benzalkonium chloride, Quaternium 12, Cocamidopropylbetaine, C9-11 Parath 8, Polyaminopropylbiguanide.

بخاخ مطهره ومنظفه لتطهير
الأسطح خالي من الكحل

Physical Properties

Clear liquid. Odour mint. pH 8-9 of solution with a shelf life of 3 years.

Packaging

1111-120 500 ml spray bottle
1111-121 5 Litre Container

Effective Spectrum

✓	Bacteria
●	MRSA
●	E-Coli
●	Pseudomonas aeruginosa
●	Enterococcus hirae
●	Staphylococcus Aureus
●	Mycobacterium tuberculosis (TB)
●	Clostridium Difficile (C-Diff)
●	Listeria
●	Salmonella typhimurium
✓	Viruses
●	HIV 1
●	H1N1 Influenza A
●	Hepatitis B Virus
●	Hepatitis C Virus (HCV)
●	Human Influenza B Virus
●	Poliovirus Type 1
●	Rotavirus
●	Norovirus
✓	Fungus
●	Aspergillus niger
●	Candida albicans
●	Penicillium

