HAMA- INSTRU™

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1111-008

1111-007

Disinfecting concentrate for instruments, burs and endodontic instruments and items.

Key Product Features

Instrument Disinfectant : Water-based disinfectant cleaner with QAC as active ingredients, with corrosion inhibitor, Concentrated, aldehyde-free and phenol-free Alcohol-free formulation, Effective against bacteria, fungi and viruses (including TB, HIV/HBV , HCV) (see table, Easily biodegradable . Safe to use, Can be used using ultrasonic device. Aldehyde and phenol free, >99%

Directions for use

2% or 4% solution. Using dosing device add 20ml to 1 litre of clean water (2%). Minimum soaking time (2%): 5 minutes (virucidal, bactericidal), 15 minutes (fungicidal). Do not exceed recommended soaking times. After disinfection, rinse instruments thoroughly with RO or distilled water. Use a 4% solution for heavily soiled items. Change the solution weekly or daily dependant on the level of use, if physical dirt is showing in the solution change sooner. Do not use on alloyed materials, malleable iron or aluminium casting parts e.g. cject syringe etc.

Product Composition

Contains amongst other ingredients: Benzalkonium chloride, Didecyldimethyl ammonium chloride, Poly(hexamethylene biguanide) hydrochloride, less than 5% Amphoteric surfactant, Disinfectants, Parfum.

Physical Properties

Blue Liquid. Pleasant Geranium aroma. pH 6.0-7.0. Water solubility: Mixable in water. Shelf life of 3 years.

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