

HAMA-JET™

DAILY

مطهر و منظف أجهزة الشفط - يومي

Daily Aspirator Cleaner and Disinfectant



**For the Daily cleaning and maintenance of Dental Aspirator Systems.
Can be used on wet line, dry line and semi wet/dry line suction systems.**

Key Product Features

Highly effective cleaning and disinfecting product. Also used for effective cleaning of the spittoon bowl. Non-foaming and non-abrasive, aldehyde, chlorinne and phenol free solution. Easy to use and measure. Colour coded. Effective against a wide spectrum of organisms. (see table opposite). Alkaline based for removal of Daily deposits.

Directions for use

Dispense 5% solution: 50ml Daily concentrate to 950ml warm water. Aspirate via the high and low volume suction tubes and also dispense into the spittoon bowl. Leave for a minimum of 10 minutes in order to activate the working properties and then rinse thoroughly with clean water.

Time of action 10-60minutes

Composition

Contains amongst other ingredients: Polyaminopropyl Biguanide, less than 5% nonionic surfactant, EDTA, Disinfectants.

Physical Properties

Blue liquid, Odour characteristic, pH 10.0 - 11.0. Soluble in water, non-flammable with a shelf life of 3 years.

Packaging

1111-010 1 Litre Container bottle
1111-009 5 Litre Container
Constrate 50 ml Equals 1 Litre
Constrate 1 Litre Equals 20 Litre
Constrate 5 Litre Equals 100 Litre

Effective Spectrum

✓	Bacteria
●	MRSA
●	E-Coli
●	Pseudomonas aeruginosa
●	Enterococcus hirae
●	Staphylococcus Aureus
✓	Viruses
●	HIV 1
●	Human Influenza B Virus
✓	Fungus
●	Aspergillus niger
●	Candida albicans



HAMA-JET™

WEEKLY

مطهر و منظف أجهزة الشفط - أسبوعي

Weekly Aspirator Cleaner and Disinfectant



1111-073

1111-022

For the Weekly cleaning and maintenance of Dental Aspirator Systems. Can be used on wet line, dry line and semi wet/dry line suction systems.

Key Product Features

Highly effective cleaning and disinfecting product. Also used for effective cleaning of the spittoon bowl. Non-foaming and non-abrasive aldehyde, chlorine and phenol free solution. Easy to use and measure. Colour coded. Acid based to remove protein and organic residues.

Directions for use

Dispense 5% solution: 50 ml weekly aspirator cleaner concentrate to 950ml warm water. Aspirate via the high and low volume suction tubes and also dispense into the spittoon bowl. Leave for a minimum of 10 minutes in order to activate the working properties and then rinse thoroughly with clean water.

Time of action 10-60minutes

Composition

Weekly Aspirator concentrate contains Citric Acid, C12-C14 Alkyldimethylammonium chloride.

Physical Properties

Green liquid, odour characteristic, pH 9-10, soluble in water, Shelf life of 3 Years.

Packaging

1111-073 1 Litre Container bottle
 1111-022 5 Litre Container
 Constrate 50 ml Equals 1 Litre
 Constrate 1 Litre Equals 20 Litre
 Constrate 5 Litre Equals 100 Litre

Effective Spectrum

✓	Bacteria
●	MRSA
●	E-Coli
●	Pseudomonas aeruginosa
●	Enterococcus hirae
●	Staphylococcus Aureus
✓	Fungus
●	Aspergillus niger
●	Candida albicans

